

# Soft Tissue Harvest



# Soft Tissue Harvest

Safe, Simple, Speedy!

Description	Type	Ref.C
Soft Tissue Harvest	L=5	TCMSTH5010

## 1. Design Concept

This is the device designed to capture the soft tissue easily when the soft tissue transplant in mouth is needed. Soft Tissue Harvester is used for structuring the soft tissue of patients.



## 2. Instruction for Use & Procedure



Find out the size and part of tissues to be collected and cut as if pulling toward the handle of the device.



After cutting the tissues as much as required length and stop the device to send back to the original place. Cutting of end part is made with #15 blade or others and collect it.

## Soft Tissue Harvest



**Head Office & Factory**  
**Gangnam Office**

45, Secheon-ro 7-gil, Dasa-eup, Dalseong-gun, Daegu, Korea  
MegaGen Tower, 607 Seolleung-ro, Gangnam-gu, Seoul, Korea

**T. +82-1544-2285**  
**T. +82-1566-2338**